
Standing Lifting poles
Bedgalg op vloerstatief
Stand-Aufrichter
Potence sur pied
Braccio di sollevamento
Incorporador de suspensión vertical

GENEVA G II V 0149 490 20

Instructions for use
Gebruiksaanwijzing
Gebrauchsanweisung
Manuel d'utilisation
Istruzioni per l'uso
Manual de utilización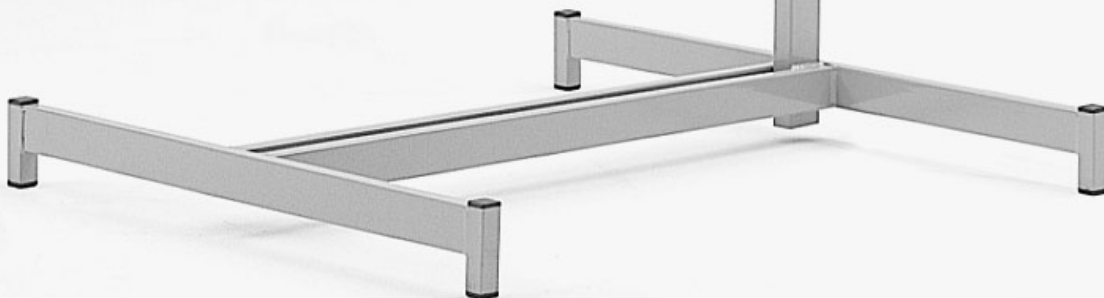

ENGLISH

We are pleased that you have decided to buy this product.

If you pay attention to some important hints, the product will ease your daily life and will be useful for years.

Please read the user's manual carefully and keep it. Pay particular attention to the following warnings.

Warnings

Ensure that the standing pole is standing safely. Use it only on even and solid ground.

Avoid too high loading to the side. For example never use for transfer from bed to arm- or wheelchair.

When using the pole at a height adjustable bed or at an adjustable chair, ensure that there is enough distance to their movable parts. Before every use check that the seams, the material of the loop and the fixing of the triangle grip are without any damage and fixed safely. Change every five years these parts for your safety (art.-no. 84011001).

Do not overload the standing lifting pole. The maximum load is 80 kg.

Range of application

The standing pole GENEVA is intended to be used indoor only.

It can be used with most of the usual beds and because of its low height of the foot part it can be further used as an aid for standing up from seats. Place the lifting pole never beside the product, but always behind the head end or back rest. Additional fixings on the wall or at the bed are not necessary.

The height of the grip is adjustable as well as the position of the grip above the bed or chair.

The standing pole can be disassembled easily, due to this fact it is easy to transport and to store.

The maximum load of the standing pole is 80 kg. This means for applications at a bed that it can be used for patients with a maximum weight of 130 kg.

If the product is not used for a longer time it has to be stored in a dry condition, protected against dust and direct sunlight.


Assembling and use

Check the intactness of the packaging at delivery. The delivering personal has to be informed immediately about damage. Later complaints about transport damage can not be accepted.

Take all parts (3 for GENEVA and 4 for GENEVA G2 with dividable pole) of the standing pole out of the packaging. All parts of the packaging are recyclable and can be disposed with normal recycling-rubbish (regard local requirements).

Place the foot on the ground.

GENEVA: Put the pole in the sleeve of the foot, so that the boom points to the foot.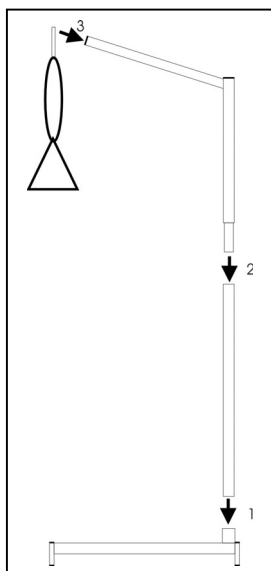


GENEVA G2: Put the straight square tube in the sleeve of the foot. Insert the bent tube into the straight square tube, so that the boom points to the foot.

Fix the triangle grip. Pay attention that the metal pin is put into a drill hole of the stand.

Regulate the height of the triangle grip to the right length. The standing pole GENEVA is ready for use now.

Position the standing pole confirm to the planned use behind the bed or chair. Make sure that the arm with the triangle grip is concentric over the patient and that the patient is not loading the standing pole sideward when using it.

The triangle grip is adjustable in the height and in its position. Therefore you have to lengthen or shorten the loop and insert the pin into another drill hole of the pole.

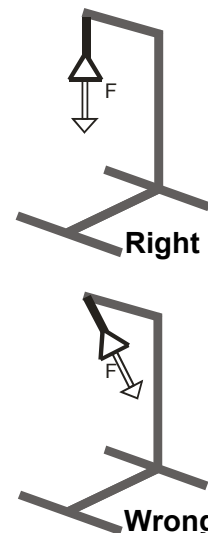Cleaning and disposal

You can clean all metal parts and the triangle grip with a mild not scrubbing detergent and wipe them with a soft cloth.

For disinfections we recommend a usual disinfectant spray.

All parts of the product are recyclable and can be disposed with normal rubbish or led to recycling.

Warranty

We take over guaranty for mistakes of production and assembling of the frame for 5 years. The guaranty starts with buying date of the product. If a part of the product is proved as defect during guaranty time, Thuasne engages to exchange this part or to repair the product. We are not liable for wrong use.

We cannot be held liable for damage or injury resulting from improper use, negligence, bad maintenance or wrong storing conditions.

Technical specifications

Height adjustable hand grip	117 – 152 cm
Overall width	89 cm
Height of the foot	10 cm
Maximum load	80 kg
Weight	15.3 kg

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